

## Measurement Pattern

Before using Coripa condoms the first time you have to determine your recommended condom size. Please follow the instructions below:

### 1ST STEP

Please cut out the two patterns along the dotted line.

Determine the breadth of your penis using **pattern I**. For this, you place the pattern around the middle of your erected penis and read off the letter code at the arrow marker.

### 2ND STEP

Determine the length of your penis using **pattern II**. For this, you keep the pattern underneath your erected penis. Pay attention to hold the dashed line at that point up to where you can unroll the condom. Read off the numerical code at the top of the glans.

### 3RD STEP

When combining the letter code and the numerical code you receive the recommended personal Coripa condom size (e.g. 15M).


The table below shows all available Coripa condom sizes.

In case your recommended condom size is not available, please advise us and select the next appropriate condom size.

### REMARK:

For most customers, the recommended condom size provides for the ideal fit of the condom. However, it may occur that in individual cases the size has to be corrected slightly. Therefore, please pay attention to a proper fit of the condom when using it.

**ATTENTION:** Please pay attention to sharp cutting edges during the application!

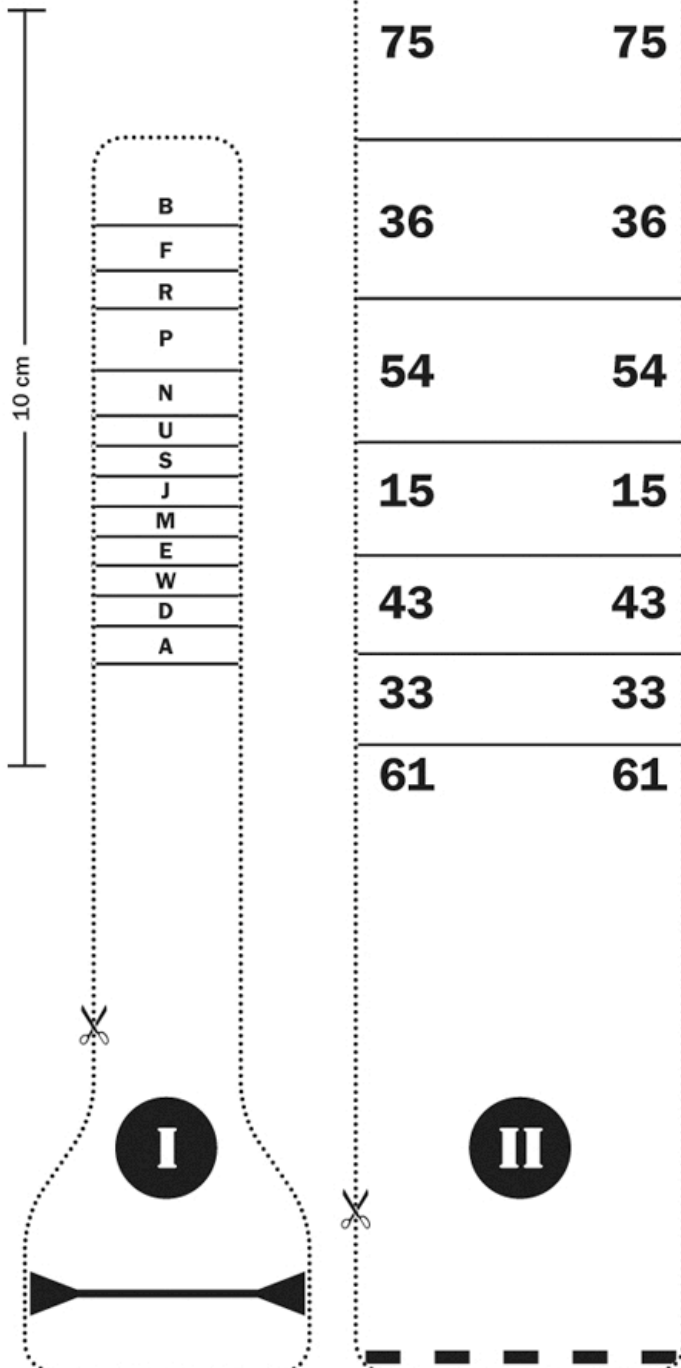

WIDE → WIDER

	A	D	W	E	M	J	S	U	N	P	R	F	B
<b>LONG</b>	61	61A	61D	61W	61E								
	33	33A	33D	33W	33E	33M							
	43	43A	43D	43W	43E	43M	43J	43S					
<b>↓</b>	15		15D	15W	15E	15M	15J	15S	15U				
	54			54W	54E	54M	54J	54S	54U	54N			
	36					36M	36J	36S	36U	36N	36P		
	75						75J	75S	75U	75N	75P	75R	
<b>LONGER</b>	29							29U	29N	29P	29R	29F	
	68									68N	68P	68R	68F 68B
	59										59R	59F	59B